

PAD PLACEMENT FOR THE BACK AND ABDOMEN



STEP 1:
Align top of pad with axilla of outstretched arm. The top of the back pad should be about a hands width below C7. Place the long side of the pad along the side of the spine.



STEP 2:
Wrap the pad from back to front, ensuring that the lines are lying anteriorly.



STEP 3:
Turn the patient and repeat on the other side, leaving a space along the spine.



STEP 4 (if needed):
For additional surface coverage, use the Universal Pad on the abdomen.

PAD PLACEMENT FOR THE LEGS



STEP 1:
Align the pad lines with the knee.
Place the short end of the pad medially and the long end of the pad laterally.



STEP 2:
Wrap the long end of the pad laterally round the leg until it is fully wrapped around the thigh.



STEP 4:
Repeat on the other side.



SPECIAL CONSIDERATIONS



LINE POSITIONS

